FR / ÉCRAN FACIAL 1188-PA19

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CAT.II **According to Regulation (EU) 2016/425 EN 166⁽¹⁾** (1) EN 166:2001 partially applied in accordance with AITEX Procedure 25DAT116 - CERTIFICATION PROCEDURE FOR FACE SCREENS AND GLASSES IN THE CONTEXT OF THE THREAT REPRESENTING COVID-19 based on Recommendation (EU) 2020/403 of the Commission of March 13, 2020 on conformity assessment and market surveillance procedures in the context of the threat posed by COVID-19.

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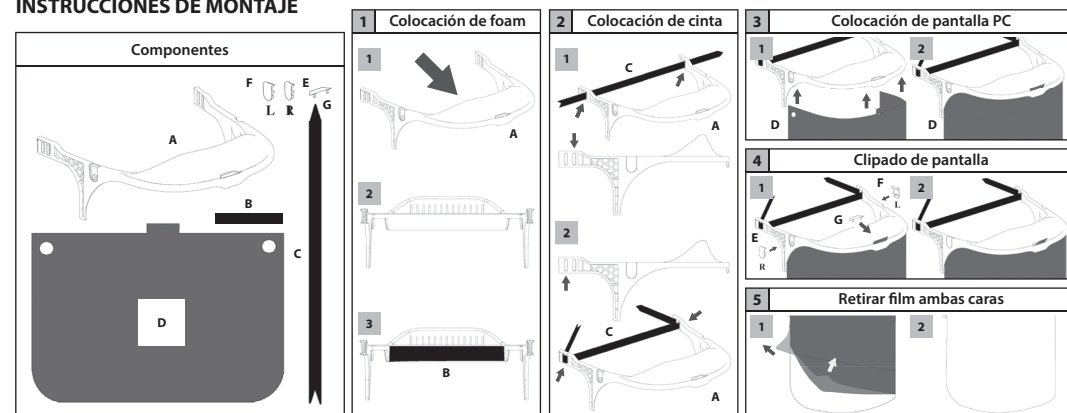
EN / FACE SHIELD 1188-PA19

CAUTION: This face shield does not provide unlimited eye / face protection. For your safety, please read these instructions completely before using this protection.

REQUIREMENTS: This face shield complies with the requirements of European Regulation (EU) 2016/425 by means of the EN 166:2001(1) standard partially applied according to the AITEX Procedure 25DAT116 - certification procedure for face shields and glasses in the context of the threat posed by COVID-19.

STORAGE AND MAINTENANCE: Keep stored in a bag and / or in a box, away from direct sunlight, high temperatures and contact with abrasives and solvents.

CLEANING, DISINFECTION AND STERILIZATION: 1. Disinfect with warm soapy water or alcohol and dry with a soft cloth or air dry protected from direct sunlight. 2. Can be sterilized by the STERRAD method.

USE: This face shield is designed to protect the user against liquid splashes in general. The visor has Optical Class 1 according to EN 166(1) so it can be used continuously. Specially certified for protection against COVID-19. This screen is reusable and ultra-light (70 g.).

MARCADO CE: Steelgen trademark. This product has been subjected to its evaluation according to the harmonized standards indicated and its conformity has been given in accordance with European legislation and it can be marketed within the European market. PPE CAT II of medium design that protects against medium risks, not being fatal or of high severity. PPE CAT II de design médio que protege contra riscos médios, não sendo fatal ou de alta gravidade.

REPLACEMENT OF SPARE PARTS AND ACCESSORIES: Use only original spare parts. To replace the screen, you must ensure that the brand is compatible and that the visor is properly installed. The user is responsible for the use of non-original accessories or incompatible with the screen.

DURATION OF USE: Inspect for damage regularly. When the face shield has become brittle or deteriorated it should be replaced immediately. The useful life of the PPE cannot be specified and it depends on the applications and the responsibility of the user to ensure that it is suitable for its intended use. Properly stored, the shelf life of this product is one year.

TRANSPORT PACKAGING: It is recommended to put a face shield (including frames and visors) in its original packaging, which consists of a plastic bag or a box in case of several units.

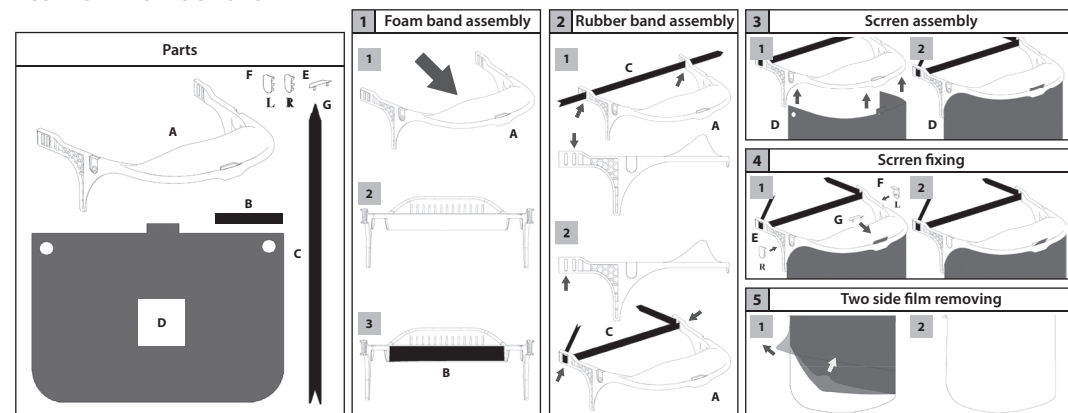
WARNINGS: 1. Do not modify or alter the face shield. 2. The pitted or scratched screen reduces vision and seriously reduces protection and should be replaced immediately. 3. This face shield has no UV, solar or IR filtering action. Does not protect against molten metal splash, hot solids, or electrical hazards. 4. Always consult with supervisory personnel to ensure that you are given adequate protection in relation to working conditions; Consult the relative European standards. 5. Materials that can come into contact with the user's skin can cause allergic reactions to susceptible people. 6. It should not be used for protection against mechanical impacts.

PLACEMENT INSTRUCTIONS: Place the screen in front of the face placing the front in the middle area so that it covers the entire face. Adjustment through the rubber located on its sides. Under foreseeable conditions of use once the screen is adjusted, it cannot be accidentally misadjusted.

CE APPROVAL: AITEX. Organismo Notificado Nº 0161. Plaza Emilio Sala, 1. E-03801 Alcoy (Alicante). SPAIN.

The EU Declaration of Conformity for this product can be found at: www.steelgen.com

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